

FITTING INSTRUCTIONS:



Step 1

Hold respirator in your hand as shown, with the nosepiece at your fingertips and the headbands hanging freely below your hand.



Step 2

Pull the top headband overhead, resting it near the top of the back of the head.



Step 3

Pull the bottom headband overhead and place around neck below ears.



Step 4

Adjust headbands for comfortable fit. Push the nosepiece inward with your fingertips to conform to the shape of your nose.

CAUTION:

The respirator should be fit-checked for each wearing. Place both hands completely over the respirator and exhale. If air leaks around nose or the edges, adjust the nosepiece and/or headbands until a good fit is achieved.