


Makrite® 910-N95




Particulate Respirator

NIOSH Approved Product



DEPARTMENT OF HEALTH & HUMAN SERVICES
U.S. GOVERNMENT PRINTING OFFICE: 2004-3323

MAKRITE INDUSTRIES INC.
11115-Syblor 7th Sec 1, Hsin Tai Wu Rd
Hsichih, Taipei Hsien, Taiwan
Tel: (02)26982419



National Institute for
Occupational Safety and Health

This respirator is only approved in the following configuration:

TC	Protection ¹	Respirator	Cautions and Limitations ²
84A-3323	N95	910-N95 x	ABCJMNP

1. Protection: N95 - Particulate Filter (95% filter efficiency level) effective against particulate aerosols free of oil; Time use restrictions may apply.

2. Cautions and Limitations

A - Not for use in atmospheres containing less than 19.5% oxygen.
 B - Not for use in atmospheres immediately dangerous to life or health.
 C - Do Not exceed maximum use concentrations established by regulatory standards
 J - Failure to properly use and maintain this product could result in injury or death.
 M - All approved respirators shall be selected, fitted, used, and maintained in accordance with MSHA, OSHA and other applicable regulations
 N - Never substitute, modify, add, or omit parts. Use only exact replacement parts in the configuration as specified by the manufacturer.
 O - Refer to users instructions, and/or maintenance manuals for information on use and maintenance of these respirators.
 P - NIOSH does not evaluate respirators for use as surgical masks.

Makrite® 910-N95



Particulate Respirator

LIMITED WARRANTY

IMPORTANT NOTICE TO PURCHASER:

This limited warranty is made in lieu of the warranties of merchantability, fitness for particular purposes and all other warranties, express or implied. There are no other warranties which extend beyond the description on the face hereof. The physical standards and specifications of Makrite will be met by products sold.

Exclusive Remedies: damages for the breach of this limited warranty are limited to the replacement of such quantity of Makrite, products proved to be defectively manufactured. Except as provided above, Makrite shall not be liable or responsible for any loss, damage, or liability, direct, indirect, incidental, special, or consequential, arising out of sale, use, or misuse, or the inability to use products by the user.

WARNINGS:

1. Wearer must read fitting instructions and use limitations before use.
2. This respirator does not protect against the risk of contracting disease or infection.
3. Individuals highly sensitive to natural rubber latex may have an allergic reaction.
4. Before use, the wearer must be trained in the proper use in accordance with applicable safety and health standards for the contaminant and exposure level in the work area.

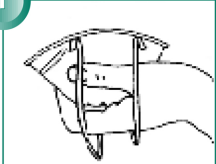
Caution:


The respirator should be fit checked for each wearing. Place both hands completely over the respirator and exhale. If air leaks around nose or the edges, adjust the nosepiece and/or headbands until a good fit is achieved.


Makrite® Industries, Inc. • Hsi-chih, Taipei Hsien, Taiwan


Makrite® Industries, Inc. • Hsi-chih, Taipei Hsien, Taiwan
Made in China

FITTING INSTRUCTIONS:

- 

Step 1
Hold respirator in your hand as shown, with the nosepiece at your fingertips and the headbands hanging freely below your hand.
- 

Step 2
Place respirator under your chin with the nosepiece up. Pull the top headband to top back of head.
- 

Step 3
Pull the bottom headband overhead and place around neck below ears.
- 

Step 4
Adjust headbands for comfortable fit. Push the nosepiece inward with your fingertips to conform to the shape of your nose.