


## TRIOSYN DISPOSABLE RESPIRATORS – T-3000 SERIES

### **DONNING and FITTING INSTRUCTIONS:**


Follow these instructions each time respirator is worn.

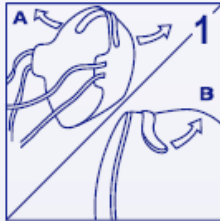

Remove respirator from package, "unfold" and pre-form the nose clip.



Adjust head straps by grasping the strap and pull adjuster until respirator fits comfortably.

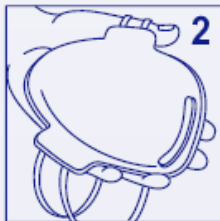

Hold the respirator in the hand, allowing the head-straps to hang loosely in front of the mask.



Fit the nose area to the shape of the face by forming the nose piece with the fingertips. Start from the top of the nose piece working downward while pressing against the nose. Always use both hands.



Position the lower head strap around the neck below the ears. Hold the respirator against the face and place the top head-strap around the crown of the head.

### **FIT-CHECK THE FACE-SEAL AS FOLLOWS:**



1. Place both hands over the respirator - do not disturb the position of the respirator.

2. Exhale sharply to detect leakage.

### **REMOVAL OF DISPOSABLE RESPIRATOR**

After use, microorganisms may be present on the exterior surface of any personal protective equipment (PPE) worn. Remove respirator and other protective apparel carefully to avoid self-contamination, utilizing appropriate hand hygiene practices immediately before and after removal. Disinfect surfaces in the immediate area where droplets of microorganisms may have fallen.