WILLSON®
SURVIVAIR BACOU-DALLOZ
 3001 S. SUSAN
 SANTA ANA - CA 92704 / USA

These respirators are approved only in the following configurations :

RESPIRATOR COMPONENTS						
TC -	Protection¹	ALTERNATE RESPIRATOR			Cautions & Limitations²	
		N9520F - S	N9520F - M	N9520F - L	N9510F - S	N9510F - M
					N9510F - L	
84A - 2883	N95	X	X	X		ABCJMNOP
84A - 2884	N95				X	X

1 - Protection

N95 - Particulate filter (95% filterefficiency level) effective against particulate aerosols free of oil ; time use restrictions may apply.

2 - Cautions and Limitations

- A - Not for use in atmospheres containing less than 19.5% oxygen.
- B - Not for use in atmospheres immediately dangerous to life or health.
- C - Do not exceed maximum use concentrations established by regulatory standards
- J - Failure to properly use and maintain this product could result in injury or death.
- M - All approved respirators shall be selected, fitted, used and maintained in accordance with MSHA, OSHA, and other applicable regulations.
- N - Never substitute, modify, add or omit parts. Use only exact replacement parts in the configuration as specified by the manufacturer.
- O - Refer to users instructions, and/or maintenance manuals for information on use and maintenance of these respirators.
- P - NIOSH does not evaluate respirators for use as surgical masks.

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WILLSON®

FITTING INSTRUCTIONS

Follow these instructions each time respirator is worn.

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Protection

- "N" series filters : effective against particulate aerosols free of oil.
- Do NOT use :**
- for protection against hazardous concentrations of gases or vapors.
- for sandblasting, paint-spray operations, asbestos.

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English

- 1 - Open the respirator by pushing slightly on both sides.
- 2 - Shape the nose-bridge.
- 3 - Place the respirator under the chin with the chin-flap fully open. Holding the respirator against the face, place the lower headstrap around the neck below the ears and the top headstrap above the ears.
- 4 - This is how the respirator should appear when correctly donned.
- 5 - Mold the nose area to the shape of the face by pinching the nose-bridge from the top to the bottom. Make sure that the chin-flap is open and secure around the chin area.
- 6 - **Fit-check the face-seal as follows :**
 - place both hands over the respirator without disturbing its position,
 - if the respirator is not fitted with an exhalation valve, exhale sharply,
 - if the respirator is fitted with an exhalation valve, inhale sharply (a negative pressure should be felt inside the respirator),
 - if you detect air-leaks, re-adjust the head-straps and/or the nose-piece.

WARNING : If you cannot achieve satisfactory fit, do NOT enter the contaminated area. See your Supervisor.

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English

USE INSTRUCTIONS :

- 1 - Failure to follow all instructions and limitations concerning the use of the respirator and/or failure to wear the respirator for the whole duration of exposure to contaminants can seriously reduce the performance characteristics of the respirator and lead to illness, injury or death.
- 2 - Before use, the wearer must first be trained by the employer in the correct use of the respirator in accordance with applicable safety and health standards.
- 3 - OSHA standards including 29CFR1910.134 (e) (5) and CSA standard Z94.4-93 require that the wearer be fit-tested.
- 4 - Discard the respirator and replace with a new one if :
 - excessive clogging of the respirator causes breathing difficulty,
 - the respirator becomes damaged.
- 5 - Leave the contaminated area if dizziness, irritation or other distress occurs.

USE LIMITATIONS :

- 1 - Do **NOT** use the respirator or enter or stay in a contaminated area under the following circumstances :
 - atmosphere contains less than 19.5% oxygen,
 - atmosphere contains oil aerosols (N95, N99, N100 only),
 - for protection against gases or vapors,
 - contaminants or their concentrations are unknown or immediately dangerous to life or health,
 - concentrations of contaminants exceed maximum use concentrations in applicable OSHA standards or applicable government regulations or 10 times the PEL (Permissible Exposure Limit), whichever is lower,
 - for sandblasting, paint-spray operations, asbestos.
- 2 - Do **NOT** modify or misuse the respirator.
- 3 - Do **NOT** use the respirator with beards or other facial hair that interferes with direct contact between the face and the edge of the respirator, or any other conditions that may prevent a good face-seal. "N" series respirators are replaced when the wearer senses an increase in breathing resistance due to the filters collecting particulate matter. Replace masks frequently.

WARNING : this respirator helps protect against certain contaminants. It may not eliminate the risk of contracting disease or infection. **Misuse may result in sickness or death.** Materials which may come into contact with the wearer's skin could cause allergic reactions to susceptible individuals.

FOR MORE INFORMATION or assistance on Willson Safety products, contact your local Willson distributor or Bacou-Dalloz.

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