



“Getting More Information”

Because the following document was developed a number of years ago, the contact information is no longer accurate. If you wish to contact the agency with questions, please use the following:

Mail: Office of Surveillance and Biometrics (HFZ 510)
1350 Piccard Drive
Rockville, MD 20850

FAX: 240-276-3356

Phone: 240-276-3357

Email: <mailto:phann@cdrh.fda.gov>

DEVICE SAFETY ALERT

HOSPITAL BEDS

June 16, 1983

Dear Hospital Administrator:

This is to alert you to a potential hazard associated with electrically powered hospital beds, and to suggest precautions you should consider to avoid accidents in the future.

Three children, ages 3, 6, and 11, have been crushed in hospital bed accidents in the past 16 months. They were caught between the stationary portion of the bed and its moving frame, following activation of automatic bed-lowering controls ("walking-away" switches). Reports alleged that the children were either playing with the controls or were curious about the beds' mechanical operation. Note that although the two U.S. fatalities were associated with Hill-Rom Model 840 beds, it is possible that other makes and models could pose a similar or problem.

It is especially important to take measures to avoid mishaps in pediatric wards and in general care areas where patient behavior may not be predictable. We suggest the following measures:

- Since this problem may not be limited to Hill-Rom beds, consider the hazard potential of all electrically powered beds in your facility, paying special attention to those which have automatic lowering devices such as the "walk-away down" feature along with a "scissors" or "guillotine" action in the metal underparts of the bed. Consider removing those beds which could be hazardous from high risk areas such as pediatric and psychiatric wards.
- If removing such beds from high risk areas is not feasible, consider deactivating "walk-away down" type switches in those areas.

Beyond such specific steps, I believe it is equally important to raise the awareness of nurses and others responsible for patient care about the potential for accidents with electrically powered beds, so that they can better anticipate problems and act to prevent injuries.

In the meantime, we are investigating the problem further with bed manufacturers and other interested organizations. We would be most interested in your experience regarding injuries associated with electrically powered hospital beds. Please direct this information to:

Dr. Joseph G. Valentino
Product Problem Reporting Program
The United States Pharmacopeia
12601 Twinbrook Parkway
Rockville, MD 20852

Toll Free number: 800-638-6725

Your cooperation is most appreciated.

Sincerely yours,

John C. Villforth
Director
National Center for Devices
and Radiological Health