

510(k) Summary of Safety and Effectiveness

Submitted By (21 CFR 807.92 (a)(1)):

Karen L. Richards
Senior Director
Chiron Corporation – Blood Testing Division
4560 Horton Street, M/S N-200
Emeryville, CA 94608-2916
Phone: (510) 923-4344
Fax: (510) 923-3703

Date Submitted (21 CFR 807.92 (a)(1)):

January 26, 2005

Device Name (21 CFR 807.92(a)(2)):

Name/Proprietary Trade Name:

Procleix[®] Optiva[™] RAS (Reagent Addition Station)

Common/Usual Name:

Procleix[®] Optiva[™] RAS

Classification Name:

Product Code: JQW

Device Classification:

Class II (Traditionally a Class I product, the RAS will be used in Blood Banking and has been deemed a Class II instrument by CBER)

This summary of 510(k) safety and effectiveness information is being submitted in accordance with the requirements of SMDA 1990 and 21 CFR 807.92.

000042

Predicate Device (21 CFR 807.92(a)(3)):

TECAN GENESIS Series RSP Model 150/8, currently marketed by TECAN Schweiz AG.

The instrument is classified under 21 CFR 862.2750 – Class I exempt, Product Code JQW. It received 510(k) clearance in 1995 (K953345) and is now 510(k) exempt.

Product Description (21 CFR 807.92(a)(4)):

The Procleix[®] Optiva[™] RAS is a bench-top system that automates the reagent dispensing steps in the licensed/registered Procleix[®] Assays. Operation of the RAS is controlled by an operator via the built-in LCD panel, which provides a user interface with the instrument firmware. RAS firmware provides automated prompts to guide operators through dispensing procedures. The RAS also features a heating element, which may be used to regulate the temperature of pipetted liquids.

RAS instruments are configured to dispense reagents from one of the following two categories:

- **Pre-Amplification:** If configured for pre-Amplification, the RAS dispenses Amplification Reagent, Oil, and Enzyme Reagent in the Amplification step of the licensed/registered Procleix Assays. The Procleix Optiva RAS is equipped with a heating block that maintains the required temperature for Enzyme Reagent addition.
- **Post-Amplification:** If configured for post-Amplification, the RAS dispenses Probe Reagent and Selection Reagent in the Hybridization Protection Assay (HPA) step of the licensed/registered Procleix Assays.

000043

Intended Use and Indications for Use (21 CFR 807.92(a)(5):

Intended Use

The Procleix[®] Optiva[™] Reagent Addition Station (RAS) is a bench top, firmware-controlled liquid pipettor intended for use in the licensed/registered Procleix[®] Assays. The RAS also features a heating element, which may be used to regulate the temperature of pipetted liquids.

Indications for Use

The Procleix Optiva RAS automates the reagent dispensing steps in the licensed/registered Procleix Assays. RAS instruments are configured to dispense reagents from one of the following two categories.

- **Pre-Amplification:** If configured for pre-Amplification, the RAS dispenses Amplification Reagent, Oil, and Enzyme Reagent in the Amplification step of the licensed/registered Procleix Assays. The Procleix Optiva RAS is equipped with a heating block that maintains the required temperature for Enzyme Reagent addition.
- **Post-Amplification:** If configured for post-Amplification, the RAS dispenses Probe Reagent and Selection Reagent in the Hybridization Protection Assay (HPA) step of the licensed/registered Procleix Assays.

000044