

CENTERS FOR DISEASE CONTROL AND PREVENTION
CHEMICAL TERRORISM BLOOD SPECIMEN COLLECTION AND SHIPPING MANIFEST

Note: Blood tubes and urine cups **cannot** be shipped together in the same package, prepare a separate shipping manifest for each. Place each shipping manifest (with specimen identification numbers) in a plastic zippered bag on top of the specimens before closing the lid of the polystyrene foam-insulated, corrugated fiberboard shipper.

Date Shipped: _____

Shipped By: _____

Name

Agency

Contact Telephone: _____

Signature: _____

Date Received: _____

Received By: _____

Signature: _____

BLOOD

Total Number of Specimens in this Container: _____

Purple Top Tubes: _____

Green- or Gray-top tubes: _____

Total Number of **Blank Tubes** in this Container: _____**Blank** Purple Top Tubes: _____**Blank** Green- or Gray-top tubes: _____

Please include two (2) empty, unopened purple-top tubes and two (2) empty, unopened green- or gray-top tubes from each lot number collected for background contamination measurement.

Place a √ in each box for samples shipped. Place an X in each box for samples not shipped. Please indicate the size of the tube collected in the comments field. Collect a minimum of 12 mL of blood. Use three 4-mL or larger vacuum-fill (unopened), non-gel, purple-top (EDTA) tubes; use four tubes if using 3-mL tubes.

PT = Purple-top tube

GT = Green- or Gray-top tube

Patient/Victim ID Label	PT 1	PT 2	PT 3	GT 1	Comments

CONTINUE ON NEXT PAGE**SHIPPING ADDRESS:**

Centers for Disease Control and Prevention
 Attn: Lt. Ernest McGahee
 4770 Buford Hwy, NE
 Building 110 Loading Dock
 Atlanta, GA 30341
 (770) 488-7579

